

Duodynator
and
Combination Duodynator-Neoserv



Instructions for use

This manual was created by gbo Medizintechnik AG and checked for accuracy. However, it does not claim to be complete. All information and data is subject to change without notice.

No part of this manual may be reproduced for any purpose without the express written permission of gbo Medizintechnik AG, regardless in which way or by any means, electronic or mechanical, are used.

© gbo Medizintechnik AG

Part-No: 032-7-0018

Edition Version: 1.3

Date of issue: 2015-02-16

Manufacturer:

gbo Medizintechnik AG

Kleiststraße 6

D-64668 Rimbach

Contents

1 INTRODUCTION.....	6
1.1 Intended use	6
1.2 Note concerning the operating personnel	6
1.3 Admission.....	6
1.4 Device description Duodynator.....	6
1.5 Device description Neoserv	7
2 START OF OPERATION	8
2.1 Unpacking, transport and installation.....	8
2.2 Combination with the Neoserv.....	8
2.3 Connect and switch on	8
2.4 Placing out of operation.....	9
3 OPERATION.....	10
3.1 Overview	10
3.1.1 Touchscreen display	10
3.1.2 Knob.....	11
3.1.3 Home screen in the single-circuit operation	11
3.1.4 Home screen panels in the interference operation.....	12
3.1.5 Screen for selecting an indication	13
3.1.6 Screen for selecting other therapies.....	13
3.2 Suction flyout.....	14
3.3 Direct help	15
3.4 Settings	16
4 THERAPY	18
4.1 General guidelines for therapy.....	18
4.2 Selection of therapies.....	18
4.2.1 Selection of therapy by Other therapies	18
4.2.2 Selection of therapy by indications.....	18
4.2.3 Selection of therapy from the favorites list	19
4.2.4 Individual settings.....	19
4.3 Implementation of the treatment.....	20
4.3.1 Treatment with interference currents.....	20
4.3.2 Therapy with single channel currents.....	20
4.4 Combination therapy with ultrasound	21
4.4.1 General information.....	21
4.4.2 Procedure for combination therapy	22
5 ELECTRODES.....	23
5.1 Electrode positioning	23
5.2 Single-pole electrodes.....	24
5.3 Suction electrodes.....	25

5.4	Bipolar electrodes.....	26
6	MAINTENANCE.....	27
6.1	Duodynator.....	27
6.2	Neoserv.....	27
6.3	Safety checks.....	28
6.4	Disposal of equipment and accessories	28
6.5	Cleaning, Disinfection and carePreventive inspection	29
6.5.1	Cleaning the unit	29
6.5.2	Cleaning of the Electrodes	29
6.5.3	Disinfecting the Electrodes.....	29
6.5.4	Cleaning of the Suction Electrodes	29
6.5.5	Disinfecting of the Suction Electrodes.....	30
6.5.6	Cleaning of the elastic straps	30
6.5.7	Cleaning of the Electrode Sponges and Electrode Pockets	30
7	WARNINGS AND HAZARDS	31
8	CONTRAINDICATIONS.....	32
9	EXPLANATION OF SYMBOLS USED	33
10	TECHNICAL DATA	34
11	ACCESSORIES DUODYNATOR.....	35
12	ACCESSORIES NEOSERV	36
13	TROUBLESHOOTING	37
13.1	Further errors	37
15	EMC HINTS	38
16	ANNEX I LIST OF LOW FREQUENCY CURRENTS.....	41
17	ANNEX II LIST OF THE MEDIUM-FREQUENCY-CURRENTS	42
18	ANNEX III LIST OF MEDIUM-FREQUENCY-PROGRAMS	43
19	INDEX.....	44

Warnings and Notes



Warning!

Warnings which must be observed!



Attention!

Read the instruction!



Note!

Information that will facilitate the work.

1 Introduction

1.1 Intended use

Therapy with low-, medium- and interference currents.

1.2 Note concerning the operating personnel

The operation of the device must be performed only by trained medical professionals.

1.3 Admission

We recommend that the operator should be trained when using the unit.

1.4 Device description Duodynator

The **Duodynator** is a microprocessor controlled electrical stimulation device for electrotherapy. The application makes the device ideal for use in the physiotherapy department of hospitals and in the modern and well-equipped practice.

The **Duodynator** is a therapeutic device for the classic electro-therapy with single-circuit currents, particularly in multi-channel operation, as well as for therapy with two-dimensional interference currents.

If two middle-frequency electrical circuits of different frequencies, or those which are phase shifted towards each other, transpose, a new frequency – the interference frequency – is generated at the intersection point through superpositioning of the superposed amplitudes. **Duodynator** uses this two-dimensional interference method. This allows the desired highest possible low-frequency stimulation from the immediate area surrounding the electrode to be shifted into deeper regions, since this is where it is generated.

Apart from the two-dimensional interference currents, **Duodynator** offers a selection of single circuit currents and provides a therapy with medium sine waves. This therapy is absolutely free of d.c. components. There are two channels available. The carrier frequency can be set from 4000 to 10000 Hz in steps of 2000 Hz.

Furthermore, the **Duodynator** offers the classical Low Frequency currents including the galvanic current (d.c.). The galvanic current is used to stimulate the circulation in the area to be treated.

The **Duodynator** thus is well suited for:

- Pain treatment
- Muscle relaxation
- Muscle training
- Muscle stimulation (increase energy consumption)
- Rehabilitation and mobilization
- Lymphatic drainage
- Edema treatment

Also the operation of the **Duodynator** is facilitated by programs, which automatically adjust the therapeutic frequencies and the treatment time.

1.5 Device description Neoserv

The **Neoserv** is a suction-wave therapy unit designed for use with 1 - 4 suction electrodes. The device is especially useful as an accessory for **Duodynator** and in the combined operation "stimulation current with ultrasound" additionally, with the ultrasound therapy device **Sonostat**.

The **Neoserv** works as suction wave massage unit. Liquids, dirt or bacteria which are sucked up by the suction electrodes are caught in a space-consuming and frequently-to-be-drained water separator.

In the areas where the suction electrodes are applied, increased hyperemia occurs. As a consequence, tissue conductivity is improved beneath the electrodes. Moreover, the patient's sensitivity to the electric current is significantly masked by the pulsating sucking action of the electrodes - a fact that makes the device particularly useful in cases where electro stimulation treatment needs to be administered to patients who are especially sensitive to electric currents.

The suction electrodes can be applied in a fast and easy manner, and they also pleasantly adapt themselves to the skin surface. Optimum skin contact is ensured by the flexible edges of the electrodes.

Because the **Neoserv** is an accessory device for **Duodynator**, which may not work alone, it is described within this manual.


2 Start of operation

2.1 Unpacking, transport and installation

After unpacking check all components for external integrity.

The unit may be placed on any flat surface. It must not be placed on the floor. Keep a wall distance of at least 20 cm. The device should neither be placed in front of radiators nor should it be covered by pillows or blankets while in operation. The device is not made for outdoor operation.

Duodynator and Neoserv meet the requirements for electrical safety according to EN 60601-1 and are both devices of protection class I. Within the scope of the Medical Devices Directive (MDD), Class IIa is assigned for the **Duodynator**. The **Neoserv** is an accessory for **Duodynator**. The combination **Duodynator-Neoserv** is in Class IIa

	<p>Warning! Note for use in the clinical area:</p> <ul style="list-style-type: none">• The unit is not designed to be operated in places with the inherent risk of explosions. If it is used in dangerous areas of anesthesia departments, the possibility of an explosion cannot be excluded.• If the patient and/or the patient cable is directly exposed to a radiator of a medical device for high frequency heat therapy, damage of the device or danger to the patient cannot be excluded. As a rule, a clearance distance of 3 m is sufficient.
--	--

2.2 Combination with the Neoserv

When the **Neoserv** is used as an accessory, **Duodynator** must be placed above the **Neoserv** and screwed on the back. Do this, slide the **Duodynator** from behind the retaining tab on the lid of **Neoserv**. Align both devices and screw the two devices on the back of each other.

In the case of the combination with **Neoserv** the **Duodynator** is supplied with mains voltage from **Neoserv**. Thus, a line input on **Neoserv** is enough to power both devices.

A detailed pictorial instruction is supplied with the **Neoserv**.

2.3 Connect and switch on

The **Duodynator** and the combination **Duodynator-Neoserv** are designed for connection to a supply voltage of 100 to 240 V and are automatically set to the correct mains voltage.


Regardless of the supply voltage, the device is suited for power frequencies of 50 to 60 Hz.



Warning!

To avoid the risk of electric shock, the **Duodynator** and also the combination of devices **Duodynator-Neoserv** must only be connected to the mains with protective earth!

Turn on the **Duodynator** by the power button on the rear panel. When **Neoserv** is connected and should also be used, it must also be turned on using the power switch on the back.

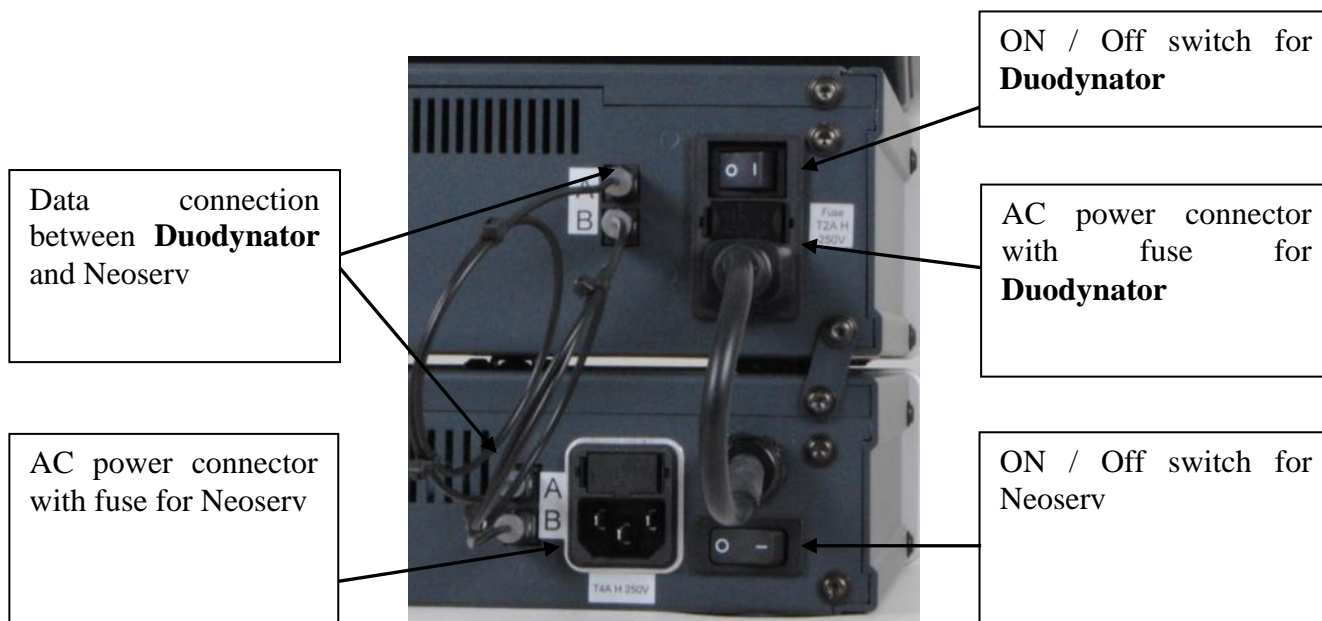

The **Duodynator** is equipped with a SD card slot, located to the right of the display. This port is intended only for service purposes.

SD card slot



2.4 Placing out of operation


In order to disconnect the device just disconnect it from the mains power supply. No other measures are to be taken.

3 Operation

The **Duodynator** has a touchscreen display, a knob, an on/off switch and a socket for a patient cable or **Neoserv**.

The operation of the **Duodynator** is done by the controls of the touch screen and the knob.

Is the **Neoserv** connected and shall be used, its operation is realized in **Duodynator**.

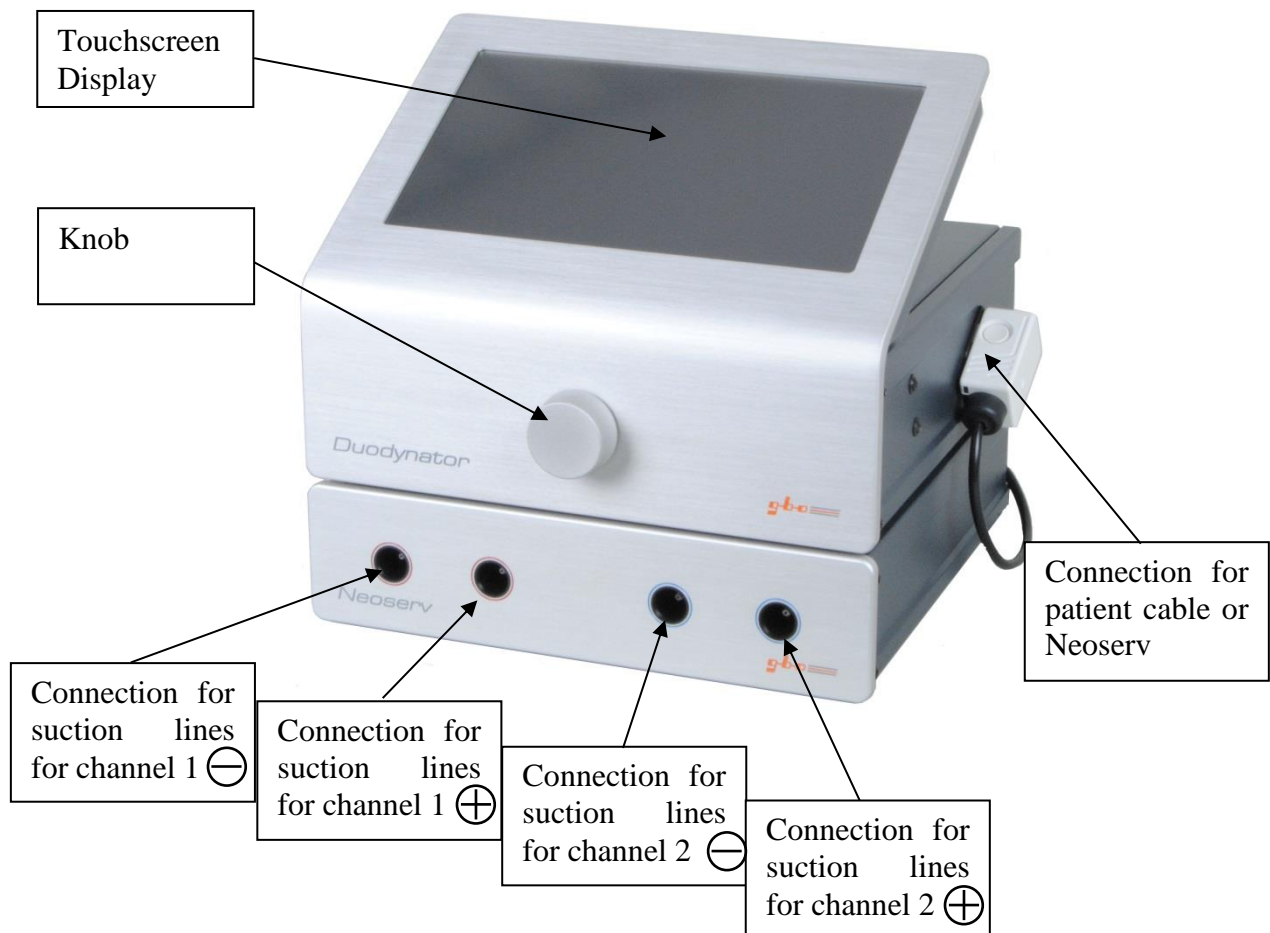

3.1 Overview

3.1.1 Touchscreen display

The touch screen should be operated with a distinct pressure of the fingertip. Sharp objects are unsuitable for operation and can damage the screen.

The individual panels (buttons) are clearly labeled. Visually accessible buttons are distinguished from non-use buttons.

3.1.2 Knob


The knob is located on the front panel and is used as an intensity knob to adjust the output voltage as well as a modification controller for changing therapy parameters such as treatment duration or frequency.

The intensity controller is activated by the selection of the corresponding channel. After a timeout of 20 seconds, the intensity controller becomes inactive and must be released for a further intensity change again with the appropriate channel button.

The Knob will be used as a modification knob after pressing a button, such as the changeable frequency. When the button is pressed, it is provided with a blue border and may be operated for 3 seconds.


3.1.3 Home screen in the single-circuit operation

After power-up, and ready for operation, the device displays the home screen.

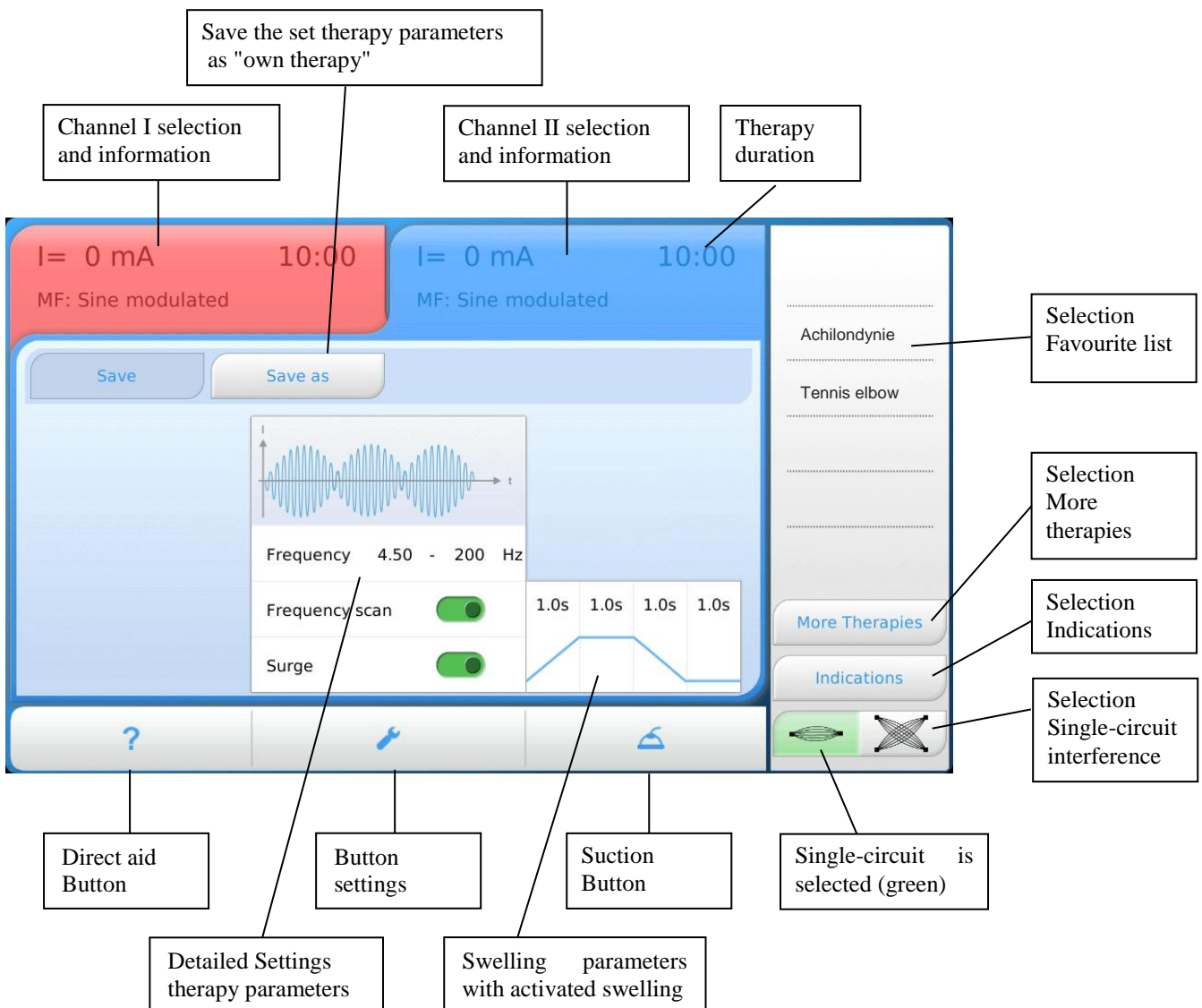

3.1.4 Home screen panels in the interference operation

After power-up, and ready for operation, the device displays the home screen.

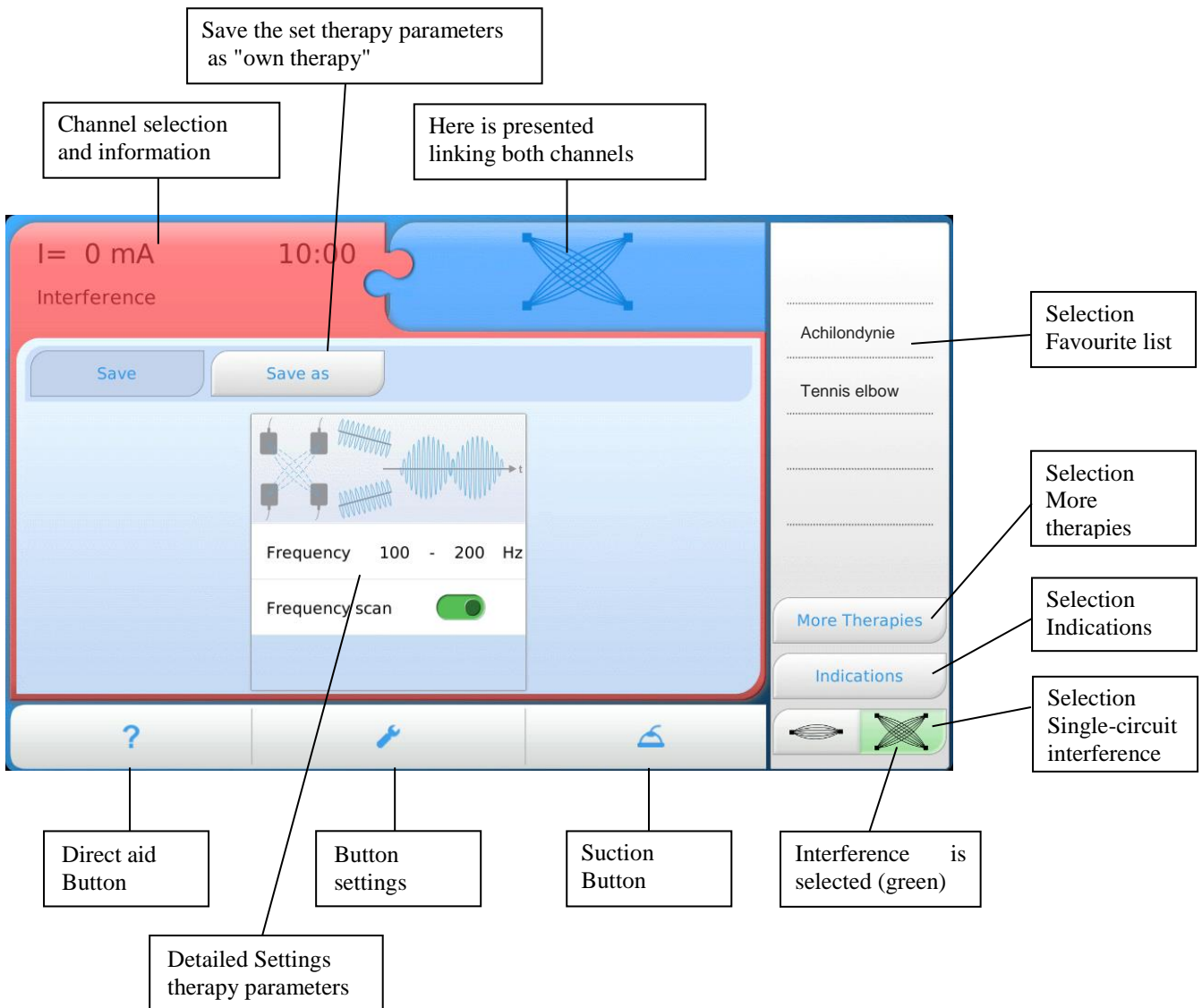

Note!
 In the interference mode, both channels are adjusted simultaneously with the intensity controller.

Note!

- If the **Neoserv** is switched on and connected to the current stimulation output of Duodynator, suction will automatically switched on when selecting a current from the Favorite list.

3.1.5 Screen for selecting an indication

After pressing the button “indications”, the device displays the following screen.

The screenshot shows the 'indications' screen. At the top, there are two status bars: a red one on the left showing 'I= 0 mA', 'MF: Sine modulated', and '10:00'; and a blue one on the right showing 'I= 0 mA', 'my therapy (MF Sine)', and '10:00'. Below these are 'Accept' and 'Cancel' buttons. The main area contains a list of indications: 'Arthrosis of the ankle joint', 'Arthrosis of the carpal region', 'Arthrosis of the knee joint', 'Arthrosis of the shoulder joint', and 'Arthrosis of the thoracic spine'. To the right of this list is a vertical range selection menu with options: 'Head / Neck / Throat', 'Upper extremities', 'Torso', 'Lower extremities', and 'All'. On the far right, there are 'More Therapies' and 'Indications' buttons. At the bottom, there are three icons: a question mark, a wrench, and a home icon. Callout boxes provide the following descriptions:

- Return to the Home screen without saving the indication (pointing to the Cancel button)
- Return to the Home screen with the acquisition of indication (pointing to the Accept button)
- Indication image Display of electrode placement (pointing to the image of a hand with an electrode)
- Indication list in alphabetical order (pointing to the list of indications)
- Range selection to restrict the list of indications (pointing to the range selection menu)

3.1.6 Screen for selecting other therapies

After pressing the button “other therapies”, the device displays the following screen.

The screenshot shows the 'other therapies' screen. At the top, there are two status bars: a red one on the left showing 'I= 0 mA', 'MF: Sine modulated', and '10:00'; and a blue one on the right showing 'I= 0 mA', 'my therapy (MF Sine)', and '10:00'. Below these are 'Change name', 'Delete', 'Accept', and 'Cancel' buttons. The main area shows a selected therapy 'my therapy' with a waveform image and parameters: 'MF: Sine modulated', '10 min', '4.50-200Hz', and '1.0s/1.0s/1.0s/1.0s'. To the right is a list of therapy types: 'LF Standard monophasic', 'LF Standard biphasic', 'MF Standard Therapies', and 'Own Therapies'. On the far right, there are 'More Therapies' and 'Indications' buttons. At the bottom, there are three icons: a question mark, a wrench, and a home icon. Callout boxes provide the following descriptions:


- Return to the Home screen without applying their own therapy (pointing to the Cancel button)
- Return to the Home screen with assumption of personal therapy (pointing to the Accept button)
- Delete the selected therapy (pointing to the Delete button)
- Rename the selected therapy (pointing to the Change name button)
- Parameters for the selected list entry (pointing to the therapy parameters)
- List of own therapies in alphabetical order (pointing to the list of therapy types)
- Pre-selection of the current type (pointing to the 'Own Therapies' option in the list)

3.2 Suction flyout

When using the **Duodynator** together with the **Neoserv** the suction operation will be activated after the **Neoserv** is switched on. Pressing the suction button, a flyout folded up in which you can set the parameters for the suction.

The suction, which means the negative pressure in the suction electrodes, can be adjusted from -40 to -300 mbar.

The suction massage can be adjusted between 0 and 20 pulses / minute.

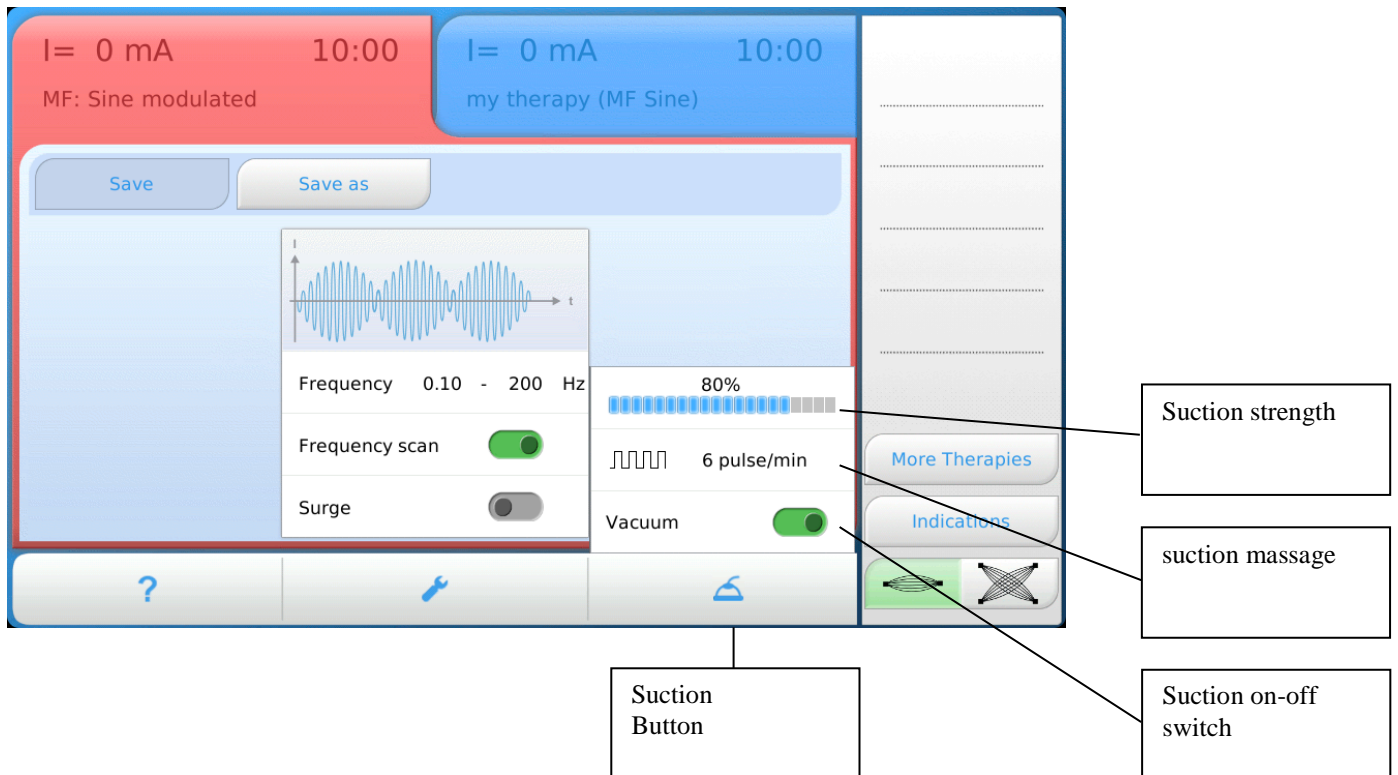

Note!

- If the suction is to be used, the **Neoserv** must be switched on with its power switch on the back of the unit before.
- If the **Duodynator** is switched off using the power switch, the **Neoserv** sounds a warning signal which indicates that the **Neoserv** is still on and must be switched off.


Note!

- If the **Neoserv** is switched on and connected to the current stimulation output of Duodynator, suction will automatically switched on when selecting a current from the Favorite list.


3.3 Direct help

The user of the **Duodynator** is guided on the screen. The direct help menu provides information about the selected buttons directly on the screen.

To open the direct help menu, first touch the “?”-button and then the button about which you desire information. A window opens where the information is shown. Touch to the help window in order to close it.

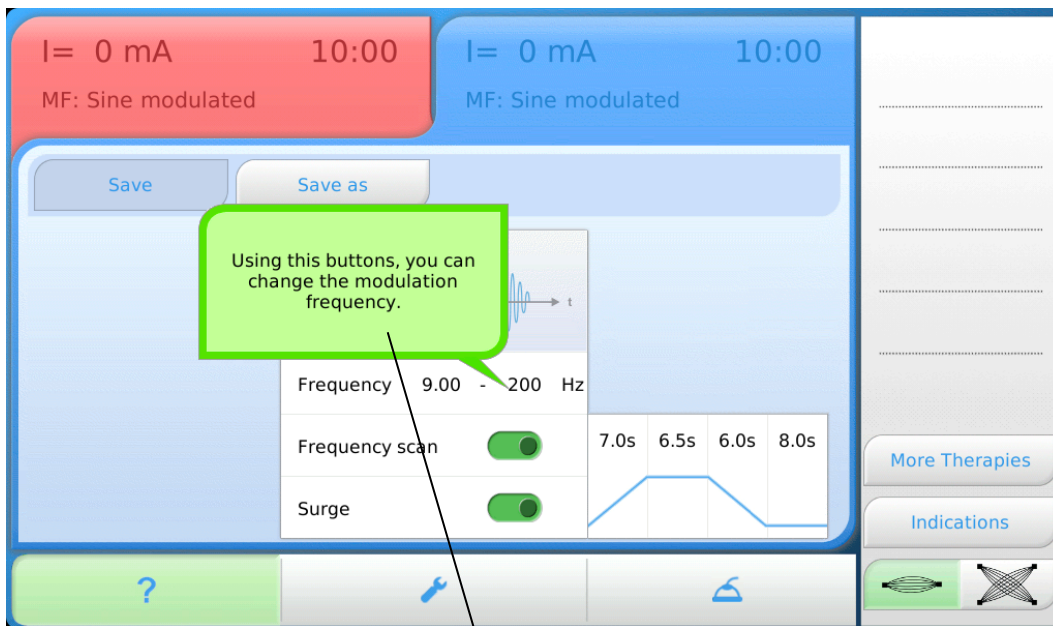

Direct Help window
Example here:
Button frequency

	<p>Note!</p> <p>Available buttons are indicated in light grey. Unavailable buttons are indicated in dark grey. If you push an unavailable button, information about the use of this button is shown on the screen automatically.</p>
---	---

3.4 Settings



Using the Settings button to access the setup menu of the **Duodynator**. In this menu, parameters such as languages, output of different sounds, eg for the end of treatment and various treatment parameters are defined.

Also the access to service is located in the settings menu.

Only if no therapy is active, you can switch to the setup menu.

	<p>Note!</p> <p>Only if no therapy is working, you can switch to the setup menu.</p>
--	---



The following items are available in the Settings menu:

Display

- Setting the national language
- Calibration of the touch screen.

Audio

- Setting the system sounds like the end of therapy-tone, error tone, etc. The sounds can be heard and assigned to the corresponding functions. A sound can also be switched off.
- Repetition of the therapy end sound (Never, 2 Minutes, 10 Minutes)
- Breaks between the repeated therapy end sound (Yes/No)
- Repetition of the error sound (Never, 2 Minutes, 10 Minutes)
- Breaks between the repeated error sound (Yes/No)

Therapy

- Setting the carrier frequency (2 to 10 kHz in 2 kHz steps)
- Temporary deactivation of the intensity monitoring until the next switching on the device (On/Off)
- Automatic pump stop after the end of treatment. (On/Off)
- Automatic switch on the pump when selecting a current from the Favorites list (On/Off)

Service

- Access to the service area

Service area with access to service. The service is reserved for service partners and helps in the search for faults in the hardware. Release of the software.

Resetting to the factory settings.

4 Therapy

4.1 General guidelines for therapy

Always observe the instructions given to the electrode positioning (see Chapter 5.1). The maximum output power per channel is 5000 mW. When this value is reached, the intensity can not be further increased. Depending on the selected current type limits for the current value can be considered (see Appendix for details).

Please note the maximum current density for the electrode which are defined in Table 1, Table 2 and Table 3.

Depending on the particular application plate electrodes or the suction electrodes can be used. When plate electrodes are used, they are connected to the patient cable.

Channel I is indicated by the red color.

Channel II is indicated by a blue color.

According to the used channel the red or blue channel must be selected on the unit to change the current for this channel.

If suction electrodes are used, a **Neoserv** must be present and connected. Instead of the patient cable, the cable of the **Neoserv** is connected to the socket of the **Duodynator**. The suction function is operated using the suction flyout. In the suction flyout the suction can be switched on and off, the suction strength can be chosen within certain limits and the suction massage frequency can be set.

4.2 Selection of therapies

4.2.1 Selection of therapy by Other therapies

A rectangular button with rounded corners and a light blue gradient background. The text "More Therapies" is centered in a blue font.

In this selection all low, all medium-and all self-defined currents are filed. The selection is made from a list which is filtered. Thus, only low-frequency monophasic, only low-frequency biphasic, only medium frequency or user-defined currents can be displayed from the appropriate list and then selected.

If Interference is used, the selection is only between standard and Own therapies.


4.2.2 Selection of therapy by indications

A rectangular button with rounded corners and a light blue gradient background. The text "Indications" is centered in a blue font.

In **Duodynator** an indication menu is integrated. This provides a list of programs with pictures indication to the electrode positioning specify the parameters (current form, possibly modulating frequency and duration). This treatment parameters are indicative and recommendations are the responsibility of the treating physician.

4.2.3 Selection of therapy from the favorites list


The Favorites list can be filled by the user as desired. Thus, the users can define the order of his frequently used therapies by himself. Indications own therapies and individual treatment settings may be transferred to the Favorites list. A simple push on the button loads the therapy from the favorite list as current therapy.

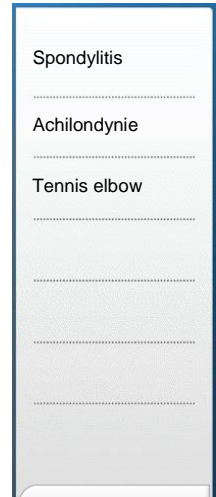



The Favorites list can be easily filled. Select a treatment from the list of indications, the list of other therapies or set the parameters individually. This therapy can now be selected by pressing and holding (≥ 2.5 seconds) the corresponding button on the right location of the favorites list. One no longer required entry can be easily deleted by push away to the right.

4.2.4 Individual settings

In the home screen there is the possibility to change parameters for a selected current waveform within certain limits. The duration can be changed accordingly. The current form as newly generated can be directly applied. It can be stored in the Favorite list. There is also the possibility to save this modified current form as "Own Therapy" under a user-specified name. These self-defined therapies can then be retrieved by the use of the button "Other therapies" with the sub-selection "Own therapies".


	<p>Note!</p> <p>After switching on the device the therapy last used will be loaded automatically.</p>
---	--

	<p>Note!</p> <p>After the change between single-circuit and interference therapies and therapies previously used will be reloaded each.</p>
---	--


	<p>Note!</p> <p>For the polarity of the connection cable applies: \oplus White cable connector or the interrupted circle on Neoserv \ominus Black cable connector or closed circle on Neoserv</p>
---	--

4.3 Implementation of the treatment

4.3.1 Treatment with interference currents

1. Turn on the power switch on the back of the unit.
2. The unit performs an automatic self-test of all functions.
3. Error free self-test is completed by displaying the Home screen.
4. Connect the accessory for interference currents to the socket on the right side, if necessary, connect the cable of the **Neoserv**.
5. Apply the electrodes to the patient (see section 5.1).
6. Press the button interference = . The last selected therapy is automatically loaded.
7. If necessary, change the therapy as described in chapter 4.2 "Selection of therapies"
8. If necessary, change the parameters and the duration for therapy.
9. Turn the intensity up slowly until the interference current is clearly perceptible, but not loosen muscular contraction duration from.
10. At the end of treatment, the Therapy-end signal sounds and the intensity automatically goes to "0.0".


4.3.2 Therapy with single channel currents

1. Turn on the power switch on the back of the unit.
2. The unit performs an automatic self-test of all functions.
3. Error free self-test is completed by displaying the Home screen.
4. Connect the accessory for single-circuit currents in the socket on the right side, if necessary, connect the cable to the **Neoserv**.
5. Apply the electrodes to the patient (see chapter).
6. Press the button single channel = . The last selected therapy is automatically loaded.
7. Select a channel button channel I or II. The selected channel is connected to the detail window.
8. If necessary, change the therapy as described in chapter 4.2 "Selection of therapies"
9. If necessary, change the parameters and the duration for therapy.
10. Turn on the intensity knob until the desired stimulus success occurs.
11. At the end of treatment, the Therapy-end signal sounds and the intensity automatically goes to "0.0".


4.4 Combination therapy with ultrasound


4.4.1 General information

The **Duodynator** may only be operated with the **Sonostat** of gbo Medizintechnik AG. When operating the **Duodynator** in combination with other devices gbo Medizintechnik AG assumes no product liability and warranty.

	<p>Note!</p> <p>If a different ultrasound device is connected, the combination have to be approved according EN 60601-1 as a system.</p>
---	---

The necessary connection between the two devices is the input jack on the back side of the Sonostat. The device's internal flow path is activated for the selected ultrasound probe.

	<p>Warning!</p> <ul style="list-style-type: none">• Observe the operating instructions of the ultrasonic therapy device (Sonostat)!• The ultrasound therapy must not be operated with the current form of electroplating due to the acidification of the contact gel.• A current density of 2 mA/cm² should not be exceeded.• Disable the current monitoring (Chapter 3.4 Settings) so that a smooth course of therapy is possible.
--	---

	<p>Note!</p> <ul style="list-style-type: none">• Combination therapy should be performed only with biphasic currents.• We recommend medium frequency currents for the combination therapy.• Use a conductive electrode gel to ensure a flow of current.
---	--

4.4.2 Procedure for combination therapy

1. Select the current form from the **Duodynator**.
2. Set the desired modifications.
3. White plugs has a positive polarity in the primary position, while the counter electrode - the metallic surface of transducer - is negative. Connect the white plug of the patient cable to the neutral electrode and the black one with the Sonostat.
4. Apply the neutral electrode.
5. Select the desired treatment head by pressing the button head left and head right.
6. Choose an own therapy, an indication or the individual setting on Sonostat.
7. Bring plenty of contact gel to the sonicated area of the patient. Move the transducer surface under light pressure to the skin.
8. Set the current intensity so that the patient experiences a significant irritant effect on the stimulation current therapy unit by turning the intensity knob. If the patient feels a pain, reduce the stimulation current intensity.
9. Press the Start button to begin the Sonostat to the ultrasonic treatment now.
10. Move the transducer surface under light pressure with circular or straight even strokes on the area to be sonicated.
11. At the end of treatment, the selected in the settings Therapy-end signal is heard. The power is automatically turned off and the ultrasonic intensity is automatically set to 0.0.
12. Clean the probe from the contact means.
13. When the treatment duration of the **Duodynator** is the elapsed, the selected in the settings Therapy-end signal and the intensity is automatically set to 0.0 sound.

5 Electrodes

The therapy can be realized without any problem with suction electrodes, conductive rubber electrodes or self-adhesive electrodes. Conductive rubber electrodes are appropriate for large-area treatments.

5.1 Electrode positioning

The positioning of the patient and/or muscle to be treated is very important. The therapy shall always be carried out in a comfortable and relaxed position. The joints shall be located in an angular position, so that they are placed in a mid-position, from which both the flexor as well as the extensor muscles can be stimulated.

In general the preparation and application of the electrodes is carried out as follows:

1. Always put a wet electrode intermediate layer or electrode pocket between the electrode and the skin in order to avoid current sensations on the skin. The layer must always be about 1 cm larger than the electrode itself.
2. Put the electrode completely into the electrode pocket, so that the pocket protrudes on the open side with about 1 cm.
3. After multiple use the electrode intermediate layer or electrode pocket can slightly shrink. If the distance of about 1 cm is no longer given, the intermediate layer or pocket must be replaced.
4. Prior to the initial use, the macro porous intermediate layers and pockets made of viscose sponge must be thoroughly washed out under running water to remove manufacturing residues.
5. To moisten the intermediate layer or pocket put them into normal tap water, or better into 1% salt solution. The intermediate layers or pockets should be soaked; remove them and let them drop; do *not* wring them out.
6. If necessary, flatten deformed electrodes, insert them into the pocket and/or put them on the intermediate layer. Put them on the area to be treated and fit them to the given body surface. Use only thick and well moistened sponges.
7. Apply the holding strap in a way that the electrode fits completely.
8. The sponge pockets or intermediate layers have to be cleaned after the expiration of treatment, *see chapter 6.5.7.*

**Note!**

Analogously, the above described also for use with suction electrodes.



Warning!

- Do not apply the electrodes on skin injuries. Even minor abrasions can cause a burning to the patient. Herewith the intensity of current by the patient can be erroneously judged. If this cannot be avoided, apply zinc ointment to cover.
- The electrodes must **adhere completely to the skin** in order to avoid **excessive local current density**. Otherwise erythemas, and sometimes even **burns may occur**.
- Current densities exceeding 2 mA/cm² require the special attention to the operator.
- It is recommended not to exceed the current density of 0,2 mA/cm² in case of galvanization and 0.1 mA/cm² in case of iontophoresis.
- If currents with galvanic portions are used special attention have to be kept. Keep care that the electrode sponges are always moistened very good.
- Do not use one way (self adhesive) electrodes in case of currents with galvanic portion, there is the danger of acid burns!!
- Pieces of jewellery and glasses have to be taken off during the treatment.

5.2 Single-pole electrodes

Single-pole electrodes are applicable for all therapy currents. For large-area treatment we recommend to use large plate or conductive rubber electrodes. Both electrode types have to be connected to the device with a patient cable. The electrode positioning is realized as described in chapter 5.1.

The electrode size depends on the area that should be passed by the current. Under small electrodes the current flow will be more concentrated and more localized than when applying large electrodes. The application area depends on the clinical picture.

Electrode	Surface	Maximum current / Galv / iontophoresis		
Rubber electrodes 8 cm x 12 cm	96 cm ²	192 mA	19.2	9.6
Rubber electrodes 11.5 cm x 17.5 cm	201 cm ²	400 mA	40	20

Table 1: Maximum currents (rms) for Rubber electrodes


5.3 Suction electrodes

Instead of the above-mentioned electrodes, suction electrodes can be used for faster application, in connection with the suction massage device. Single-pole suction electrodes are applicable for all therapy currents. For this please plug in the connection cable of the **Neoserv** to the socket of the current stimulation device **Duodynator**.

Suction electrode	Surface	Maximum current
Electrode plate Ø 60 mm with sponge Ø 60 mm	28 cm ²	56 mA
Electrode plate Ø 90 mm with sponge Ø 90 mm	63.5 cm ²	127 mA

Table 2: Maximum currents (rms) for suction electrodes

5.4 Bipolar electrodes

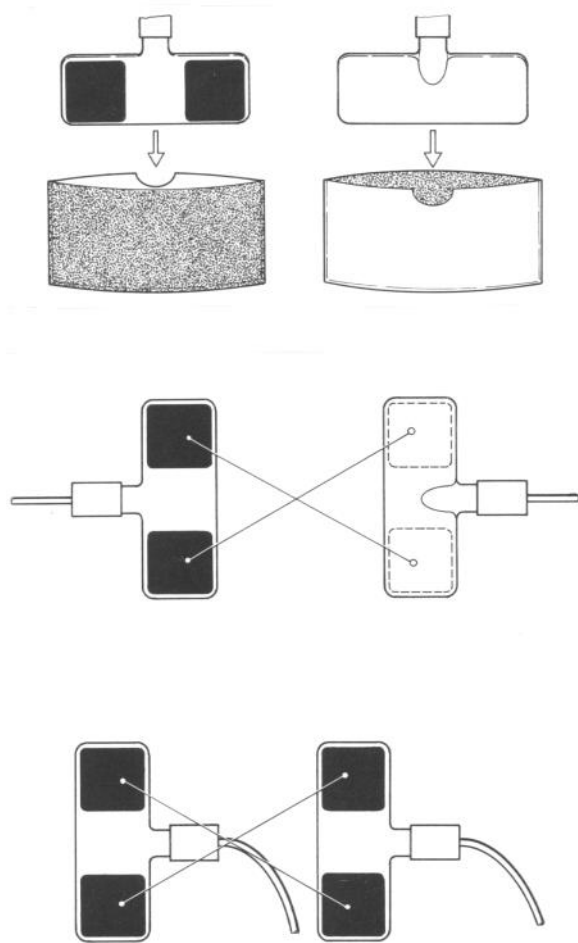

The bipolar electrodes can be used for the duodynamic interference currents.

The conductive rubber electrodes of the two current circuits are located in two flexible bipolar duo electrode (I and II) and attached to a common patient cable. There are two sizes available (small and standard). Electrode pockets are necessary for application and fixing material for bandage (see accessories).

Each bipolar electrode is placed into a well wetted pocket so that the black conductive rubber electrodes face the humidity carrier. The reverse side of the pocket consists of transparent synthetic material in order to be able to check the correct positioning of the electrode.

Electrode position:

- A) If the electrodes are placed opposite to each other (**transverse current flow, transregional**), the cable connections must point in opposite directions.
- B) If the electrodes are placed on one side (**longitudinal current flow**), their cables must point in the same direction.

Only in the case of these electrode positions the flow paths of current of the two circuits can transpose and optimal duodynamic stimulation currents are generated.

In special cases (e.g. depending on the application in the case of shoulder-hand syndrome) it is possible to deviate from this general rule.

Electrode	Surface	Maximum current
Duo-Electrode small	12 cm ²	24 mA
Duo-Electrode large	27 cm ²	54 mA


Table 3: Maximum currents (rms) for Duoelektroden

6 Maintenance

6.1 Duodynator

Functionality, reliability and safety characteristics of the **Duodynator** are guaranteed only upon handling the device in accordance with the operating instructions. Safety control, maintenance work, repair work and modifications must only be carried out by the manufacturer or by service agents authorized by him. In case of a failure, parts which influence the safety of the device must only be replaced by original spare parts of the manufacturer. The electric installation must correspond to the requirements in accordance with VDE/IEC. **The device does not contain any parts which need maintenance work done by the user.**

6.2 Neoserv

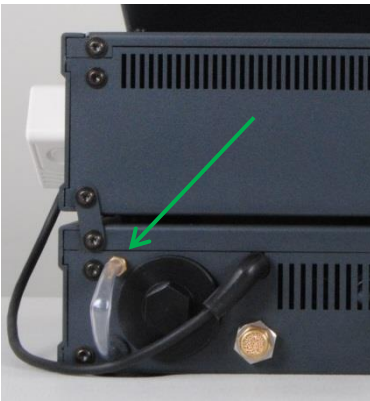

The **Neoserv** is a suction application device. An under pressure is generated in the device which sucks the air from electrodes into the device. In the device a water-and dirt separator keeps liquids away from the pump liquids. This container is provided with a water level control. When filling the container beyond a certain limit, a message is displayed which recalled that the container needs to be drained. Hold a container ready and pull the hose to empty the container at the back of the device from the lock nipple. Turn the hose down and let the liquid of the container drain into the receiving vessel. Is all the liquid out of the container, put the hose back on the sealing nipple.

At the annual safety inspection, the service technician should clean the container. This is accessible from the rear of the **Neoserv**. First drain the tank as described above. Then tilt it slightly backwards and unscrew the container. You can now clean it under running water and screw back into the machine.

6.3 Safety checks

The device is subject to the provisions of the Medical Device Directive. The safety controls have to be carried out on the basis of this directive. Thereby, the operator regulation has to be observed in particular.

Irrespective of the legal rules or beyond the scope of the Medical Device Directive, it is mandatory to have the device checked at 12-months intervals by the manufacturer or by a maintenance service authorized by him.

The review must consist at least of the following criteria:

- Electrical safety check in accordance with the test plan of the manufacturer
- Review of the device in respect of external integrity
- Review of all display and operating elements in respect of damages
- Review of all inscriptions in respect of immaculate legibility
- Review of power supply cord and all patient connecting cables
- Functional check

6.4 Disposal of equipment and accessories

According to the WEEE Directive 2002/96/EG (waste electrical and electronic equipment) this device must not be disposed of with the domestic waste. The device must be returned to the manufacturer for disposal. The manufacturer is committed to guarantee the disposal of all devices marketed. This is also indicated by the WEEE sign (crossed out waste container).

6.5 Cleaning, Disinfection and care Preventive inspection

6.5.1 Cleaning the unit

Clean the device with a clean, lint-free cloth dampened with water and mild detergent. Please do not use abrasive cleaners. Fingerprints on the display can be removed with a dry cloth or isopropanol.



Warning!

- Do not hold the device under running water and do not use liquid cleaners!
- Before cleaning or disinfecting always pull out the mains plug from the wall outlet!

The **Duodynator** is suitable for wipe disinfection. It is important to ensure that no liquids get into the unit. Do not wet plugs or sockets. The device must not be sprayed for cleaning or disinfecting. The device and its accessories are not suitable for heat sterilization or sterilization with gases.

6.5.2 Cleaning of the Electrodes

You can clean the electrodes after a treatment by using some warm water (appr. 40 °C) and a soap suds on the conductive (black) side. Dry the electrodes with a cleaning tissue or let them air-dry. The cleaning process removes the film left by the Aloe spray or the contact spray and the electrodes will regain a good conductivity.

6.5.3 Disinfecting the Electrodes

The rubber electrodes can be disinfected by spray/wipe disinfection. Spray onto the black side of the electrodes, leave to soak and wipe off with a lint free cloth. (Please follow the recommendations of the manufacturer.)

We recommend Bacillol-AF for the disinfection of the electrodes.



Note!

The conductivity of the electrodes is optimized by the use of a certain amount of graphite. Consequently, when using and disinfecting the electrodes, black color may come off.

6.5.4 Cleaning of the Suction Electrodes

You can clean the electrodes after a treatment by using some warm water (appr. 40 °C) and a soap suds. Dry the electrodes with a cleaning tissue or let them air-dry.

6.5.5 Disinfecting of the Suction Electrodes

The suction electrodes can be disinfected by spray/wipe disinfection. Spray onto the black side of the electrodes, leave to soak and wipe off with a lint free cloth. (Please follow the recommendations of the manufacturer.)

We recommend Bacillol-AF for the disinfection of the electrodes.

6.5.6 Cleaning of the elastic straps

The elastic straps can be washed in a washing machine. Please observe the washing instructions of the sewed label or the instructions on the label of the packaging.

6.5.7 Cleaning of the Electrode Sponges and Electrode Pockets

The Electrode sponges and pockets can be washed in a washing machine with 95 °C. or be disinfected with disinfectant solution.

We recommend Dismozon[®] pur (chemical agent: Magnesium monoperoxyphthalat Hexahydrat).



Note!

- After disinfection the sponges and pockets have to be wringed out closely at least 6 times under warm running water!

7 Warnings and hazards



Warning!

- For patients with implanted electronic device electrical stimulation treatment is to be carried out only after having checked any risks.
- Turn off cellular phones and radiophones or place them in a distance of 3 m from the device.
- Cardiac pacemakers are extremely vulnerable. Here the therapy should only be carried out under continuous pulse and ECG control.
- If the patient and/or the patient cable is in the direct range of a high-frequency, short-wave or micro-wave therapeutic device, a damage to the device or an injury to the patient cannot be excluded. Please keep a clearance distance of 3 m.
- A simultaneous connection of the patient to a high-frequency surgery device can lead to burns under the electrical stimulus electrodes.
- The unit is not designed to be operated in places with the inherent risk of explosions. If it is used in dangerous areas of anesthesia departments, the possibility of an explosion cannot be excluded.
- Current densities exceeding 2 mA/cm² require the special attention to the operator.
- It is recommended not to exceed the current density of 0,2 mA/cm² in case of galvanization and 0.1 mA/cm² in case of iontophoresis.
- If currents with galvanic portions are used special attention have to be kept. Keep care that the electrode sponges are always moistened very good.
- Do not use self-adhesive electrodes at currents with galvanic portion; they contain corrosive!
- The electrodes must be (fully covered) rest with the entire electrode surface in order to avoid local current density peaks. Otherwise, skin redness may occur in severe cases even burns.



Note!

In case of any visible failure contact gbo Medizintechnik AG or one of the service agencies authorized by gbo Medizintechnik AG immediately.

8 Contraindications

Despite the low dosage is definite contraindications have emerged.



Warning!

Stimulation should not be applied in the following syndromes:

- Open wounds in the treatment area
- Malignant tumors in the treatment area
- Feverish conditions

Stimulation current to be applied to the following symptoms only after consultation with the doctor:

- Patients with implanted electronic devices
- Patients with metallic prostheses (only biphasic currents should be used, preferably medium-frequency current used)
- Pregnancy



Additional precautions!






- In patients with diagnosed heart problems
- In treatment on areas of the skin which lack normal sensation assets
- For muscle stimulation after recently running operations in which muscle contraction may disrupt the healing process



Note!

- This list is not exhaustive. In individual cases the attending doctor should decide on contraindications and criterias for the treatment.
- In case of all visible failures contact gbo Medizintechnik AG or one of the service agencies authorized by gbo Medizintechnik AG immediately.

9 Explanation of symbols used

	CE - Conformity mark
	Read the instruction!
	Caution!
	Application part of Type BF (floating).
	This product complies with WEEE Directive 2002/96/EG (waste electrical and electronic equipment). Separate collection for electrical and electronic equipment. The waste removal at the end of the service life will be done by the manufacturer.

10 Technical Data

Mains voltage and Frequency:	Duodynator: 100 - 240 V / 50-60 Hz Neoserv: 100 - 240 V / 50-60 Hz
Power consumption:	Duodynator: Max 65 VA Neoserv: Max. 105 VA (incl. Duodynator)
Main fuses:	Duodynator: T 2A H 250V Neoserv: T 5A H 250V
Max output Power	5000 mW = 100mA to 500 ohms per channel
Accuracy of values displayed	Current: + / - 10%
Permissible load impedance	30 Ω 8 kΩ
Suction Neoserv	40-300 mbar
Suction massage Neoserv	0-20 pulses / min
Applied part	Contact surfaces at the electrodes.
Protection	IP X7 (Duodynator and combination)
Mode of operation:	Continuous operation (Duodynator and combination)
Protection class:	I
Degree of protection:	BF (Duodynator and combination)
Protection class of housing:	IP X0 (Duodynator and combination)
Dimensions:	Duodynator 27 cm × 19.3 cm × 28 cm (W × H × D) combination Duodynator and Neoserv 27 cm × 27 cm × 28 cm (W × H × D)
Weight (without accessories):	Duodynator: 4,4 kg Combination Duodynator-Neoserv: 7.7 kg
Display:	10.1 "TFT LCD with touch screen
Environmental conditions:	Operation of the device: Temperature range +10 ° C ... +35 ° C relative humidity 30 ... 75%
	Transport and storage: Temperature range -10 ° C ... +50 ° C relative humidity <90%, non-condensing

By request of technical personnel gbo Medizintechnik can offer spare part lists and circuit diagrams.

The IEC connector is used for all-pole disconnection from the mains.

gbo Medizintechnik AG reserves the right to change design and specifications without prior notice.

11 Accessories Duodynator

Included accessories:

032-0-2000	1	Power cord (Black)	Article no. 027-4-6001
	1	User manual	032-7-0017

Other accessories for the **Duodynator**:

Description	Article no.
Therapy set Duodynator (connection cable, 4 conductive rubber electrodes 8 x 12 cm, 10 sponge bags, 4 Velcro)	032-0-2020
Duo-electrode set, standard (Duo electrode pair standard, electrodes bags, rubber bands)	45-38-310EH720
Duoelektroden pair, small	45-38-781EH725
Duoelektroden-pair standard,	45-38-773EH725
Electrodes bag, small (10 pieces) for Duoelektrode small	45-38-807EH725
Electrodes bag, standard (10 pieces) for Duoelektrode standard	45-38-799EH725
Connection cable for Duodynator	032-0-2010
rubber electrodes 8 cm x 12 cm (2 pieces)	017-0-0047
rubber electrodes 11.5 cm x 17.5 cm (2 pieces)	017-0-0046
Sponge bag for rubber electrodes 8 cm x 12 cm (10 pieces)	45-38-203EH725
Sponge bag for rubber electrodes 11.5 cm x 17.5 cm (2 pieces)	020-453813
Elastic strap with velcro blue (2 pieces)	011-0-0033
Electrode handle with ball and disc electrode	014-0-0088
Adapter cable for handle electrode	017-0-0064
Device cart basic rack	026-0-3000
Mounting kit Duodynator (Equipment storage and electrode holder)	026-0-3030

Recommended accessories are highlighted in grey!



The picture shows the device cart for the **Duodynator**

It consists of the basic rack and the mounting kit **Duodynator**


12 Accessories Neoserv

032-0-4000	1	Power cord (Black)	Article no. 027-4-6001 032-7-0019
	1	Installation Instructions	

Other accessories for the **Neoserv**:

Description	Article no.
Standard suction electrodes set (2 x electrode hose red, 2 x electrode hose blue, 4 x suction electrode Ø 60 mm, 4 x contact sponge)	032-0-2110
Electrode hose red, 1750 mm long	014-0-0050
Electrodes blue hose, 1750 mm long	019-0-0053
Suction electrode, Ø 60 mm	014-0-0083
Suction electrode, Ø 90 mm	014-0-0084
Contact sponge, Ø 60 mm (4 pieces)	014-0-0086
Contact sponge, Ø 90 mm (4 pieces)	014-0-0087
Short-circuit hose (when using only one channel)	014-0-0026
Connection cable "Neoserv ultrasound"	014-0-0014

Recommended accessories are highlighted in grey!

	<p>Note!</p> <ul style="list-style-type: none"> • Use only original accessories to ensure the safe operation of the appliance.
---	--

13 Troubleshooting

Each problem that occurs during operation of the unit will be shown in a message window on top of the display and also signaled by an acoustic tone. Most of the problems can be solved by the instructions displayed.

In general:

1. The malfunction is shown on the display.
2. An acoustic signal is heard.
3. Follow the instructions on the display.


Suggestions:

- Turn off and turn on the unit.
- A full selftest of all device functions is performed and the unit is reinitialized. If the unit does not come up to normal state, please call your local service.

13.1 Further errors

symptom	cause and action
Device cannot be switched on, no display shown.	Please check the main plugs and sockets. If necessary; contact your service agents or the manufacturer.
No acoustic signal is heard. (End of therapy...)	Please check the settings of the acoustic tones in the menu. The volume must be greater than 0.

Please contact your service agent or the manufacturer if the problems cannot be solved by the measures mentioned above.

	<p>Note!</p> <p>Please note that the unit must be placed on a plane horizontal surface. The device should neither be placed in front of radiators nor should it be covered with pillows or blankets while in operation. Do not cover the ventilation slots on the bottom of the unit either.</p>
---	---

15 EMC Hints

Comments According to the Medical Device Directive

The **Duodynator** is a mains operated current stimulation device of protection class **I**.

The device is in accordance with the EC Medical Device Directive (93/42/EEC) and therefore carries the CE-sign with the number of the "notified body for medical devices". The respective graphical symbol is placed on the type plate.

According to the Medical Device Directive, **Duodynator** is a device of class **IIa**.

The manufacturer is only responsible for the security, operational reliability and functionality of the device if:

- * the device is used in accordance with the user manual;
- * the electrical installation of the location where the device will be used corresponds to the respective current requirements of electrical safety;
- * the device is not used in hazardous environments and humid locations;
- * the mountings, add ons, internal adjustments, modifications or repairs are realized only by personnel authorized by the manufacturer;
- * the operator regulation of this EC-directive is observed within the scope of the Medical Device Directive.

You may obtain technical support by the manufacturer or the dealers or service authorized by the manufacturer. The manufacturer projects a product life of 7 years.

Duodynator is an electronic device. Disposal has to be done according to regulations for electronic devices. Consumables have to be disposed as residual waste.

On request, the manufacturer will provide you with further technical descriptions for all serviceable parts of the device, such as circuit diagrams, spare part lists and adjustment instructions as far as these are of use for the qualified technical staff of the user.

Comments on electromagnetic compatibility (EMC)

Medical, electrical devices are subject to special precautions concerning the EMC. They must be installed and operated according to the EMC-advice given in the accompanying documents. In particular medical, electrical devices may be influenced by portable and mobile RF-communication devices.

The manufacturer guarantees the conformity of the unit with the EMC-requirements only when using accessories which are listed in the EC declaration of conformity. The usage of other accessories may cause an increased emission of electromagnetic disturbances or may lead to a reduced electromagnetic immunity.

The unit must not be arranged physically close to other devices or stacked with them. If such an order is necessary nevertheless, the unit must be observed in order to check it for the intentional operation.

You find more EMC-comments in the chapter "Warnings and Safety Precautions" of this manual as well as in the Technical Information on the next two pages.

In accordance with the EMC-regulations for medical products we are obliged by law to provide the following information.


Guidance and manufacturer’s declaration — electromagnetic emissions

The equipment is intended for the use in the electromagnetic environment specified below. The customer or the user of the equipment should ensure the compliance with this stipulation		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions, CISPR 11	Group 1	The equipment uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The equipment is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions, CISPR 11	Class B	
Harmonic emissions, IEC 61000-3-2 (*)	Class A	
Voltage fluctuation/flicker emissions, IEC 61000-3-3 (*)	complies	
(*) Note: For devices with a power consumption between 75 W and 1000 W only.		

Guidance and manufacturer’s declaration — electromagnetic immunity

The equipment is intended for the use in the electromagnetic environment specified below. The customer or the user of the equipment should ensure the compliance with this stipulation.			
Immunity test	IEC 60601- test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD), IEC61000-4-2	±6 kV contact	±6 kV contact	Floors should be made of wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
	±8 kV air	±8 kV air	
Electrical fast transient/burst, IEC 61000-4-4	±2 kV for power supply lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
	±1 kV for input/output lines	±1 kV for input/output lines	
Surge, IEC 61000-4-5	±1 kV differential mode	±1 kV differential mode	Mains power quality should be that of a typical commercial or hospital environment.
	±2 kV common mode	±2 kV common mode	
Voltage dips, short interruptions and voltage variations on power supply input lines, IEC 61000-4-11	<5% U _τ for ½ cycle (>95% dip)	<5% U _τ for ½ cycle (>95% dip)	Mains power quality should be that of a typical commercial or hospital environment. If the user of the equipment requires continued operation during power mains interruptions, it is recommended that the equipment be powered from an uninterruptible power supply or a battery.
	40% U _τ for 5 cycles 60% dip)	40% U _τ for 5 cycles 60% dip)	
	70% U _τ for 25 cycles 30% dip)	70% U _τ for 25 cycles 30% dip)	
	<95% U _τ for 5 s (>5% dip)	<95% U _τ for 5 s (>5% dip)	
Power frequency (50/60 Hz) magnetic field, IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Note: U _τ is the a.c. mains voltage prior to application of the test level.			




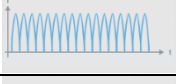
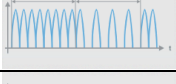
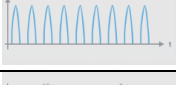
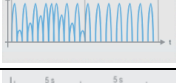
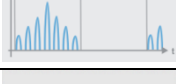
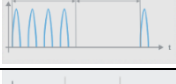

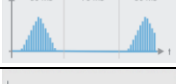
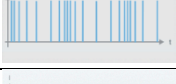






Guidance and manufacturer's declaration — electromagnetic immunity

The equipment is intended for the use in the electromagnetic environment specified below. The customer or the user of the equipment should ensure the compliance with this stipulation.			
Immunity test	IEC 60601- test level	Compliance level	Electromagnetic environment – guidance
			Portable and mobile RF communications equipment should be used no closer to any part of the equipment, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance:
Conducted RF, IEC 61000-4-6	3 V _{rms} 150 kHz to 80 MHz	3 V _{eff}	$d=1.2\sqrt{P}$
Radiated RF, IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3V/m	$d=1.2\sqrt{P}$ for 80 MHz to 800 MHz $d=2.3\sqrt{P}$ for 800 MHz to 2.5 GHz
			Where P is the maximum output power rating of the transmitter in watts according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Interference may occur in the vicinity of equipment marked with the following symbol: 

Recommended separation distances to portable and mobile RF communication equipment

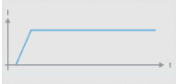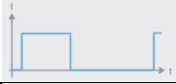The equipment is intended to be operated in an electromagnetic environment where radiated RF interference is controlled. The user can help to avoid interferences by meeting the minimum separation distances between portable and mobile RF communication equipment (transmitters) according to the maximum output power of the communication equipment.			
Rated power of the transmitter (W)	Separation distance according to the transmission frequency (m)		
	150 kHz to 80 MHz $d=1.2\sqrt{P}$	80 MHz to 800 MHz $d=1.2\sqrt{P}$	800 MHz to 2.5 GHz $d=2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

16 Annex I List of low frequency currents



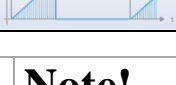
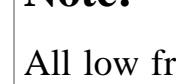

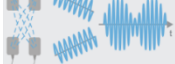
Current form	Graphic	Pulse duration in ms	Pause duration in ms	DC portion in%	Frequency in Hz	Maximum Voltage Vpeak	Maximum Current in mApeak
Galvanic current		-	-	100	0	100	50
Ultra Stimulation		2.0	5.0	28.6	143	100	70
Faradic current		1.0	19.0	2.5	50	100	100
DF Diphase fixed		-	-	63	100	100	70
CP Courtes périodes		-	-	47.25	50/100	100	70
MF Monophasé fixed		10	10	31.5	50	100	70
LP Longues périodes		-	-	46	50/100	100	70
MM Monophasé modules		-	-	20	100	100	70
RS Rythme syncope		-	-	15.25	50	100	70
Impulse galvanization IG 30		30.0	50.0	18	12.5	100	70
Impulse galvanization IG 50		50.0	70.0	0.3	8.3	100	70
Frequency modulation FM / 7 - 14 Hz		0.5	70.9 to 142.4	0.4	7-14	100	100
Surged current aS		-	-	32	0.2	100	70
Surged current aSL		-	-	45	00:09	100	70
Tens		0.2	4.8 to 666	<3.5	1.5 to 200	100	100
Tens burst		1	4-666	<10	1.5 to 200	100	100
E100		100,0	500,0	7,14	1,67	100	100
E200		200,0	1000,0	7,14	0,83	100	100



Note!

All low frequency currents except Galvanic current are also available as metal- acceptable biphasic variant.

17 Annex II List of the medium-frequency-currents

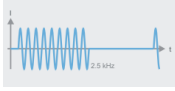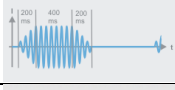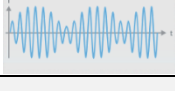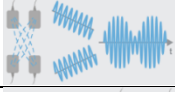Current form	Graphic	Pulse duration T in ms	Pause duration R in ms	DC share in %	Frequency in Hz	Maximum Voltage V _{peak}	Maximum Current in mA _{peak}
single-circuit							
Russian stimulation (Kots)		10	10	0	50	100	100
Gym Current		800		0	0,25 - 0,5	100	100
MF sinus-modulated				0	0.1 to 200	100	100
double circuit							
Interference				0	0.1 to 200	100	100
Dipole vector				0	0.1 to 200	100	100
Isoplanar Vector				0	0.1 to 200	100	100

18 Annex III List of medium-frequency-programs

Program	Step 1	Step 2	Step 3	Remark
Electro massage	0.5 to 3 Hz			1.5 Hz * Adjustment phase
Shaking massage	3-10 Hz			
Sports	20 Hz 5 s / 5 s / 20 s			
Rehab	0.2 to 0.5 Hz			1.5 Hz * Adjustment phase
Mobilization	0.5 Hz 14 s	1-8 Hz 22 s		1.5 Hz * Adjustment phase
Muscle pump	0.3 Hz			1.5 Hz * Adjustment phase
Trauma	10 Hz 1/2 Time.	100 Hz 1/2 Time		
Lymphatic drainage	0.3 Hz 10 min	10 Hz 10 min	10-30 Hz 10 min	
Edema	10 Hz			
Pain	90-110 Hz 1 s / Freq.			
Universal	0.1 to 200 Hz 1 s / Freq.			1.5 Hz * Adjustment phase

* Adjustment phase 4 seconds



Note!

This list also applies also for interferential currents.
Currents for muscle stimulation are excluded in the Interference treatment.

19 Index

- A
- Accessories Duodynator 35
Accessories Neoserv 36
- B
- Bipolar electrodes 26
- C
- care 29
Cleaning 29
Combination therapy 21
Connect and switch on 8
Contraindications 32
- D
- Desinfection 29
Device description Duodynator 6
Device description Neoserv 7
Direct help 15
- E
- Electrode positioning 23
Electrodes 29
EMC Hints 38
- H
- Home screen 11
Home screen interference 12
- I
- indications 13
installation 8
Intended use 6
- K
- Knob 11
- L
- low frequency currents 41
- M
- mains voltage 8
Maintenance 27
medium- frequency-currents 42
medium-frequency-programs 43
- O
- Operation 10
- P
- polarity 19
precautions 32
protective earth 9
- S
- Safety checks 28
SD card 9
Settings 16
Single-pole electrodes 24
suction electrodes 25
Suction flyout 14
symbols 33
- T
- Technical Data 34
Therapy 18
transport 8
Troubleshooting 37
- W
- Warning 21
warnings 8, 31
Warnings 24

